




DENTO-VIRACTIS ASPIRATION 99

Dilution:
2%

DENTO-VIRACTIS

Description	<p>Dento-Viractis 99 is a deodorizing cleaning liquid for treating surgical aspiration systems, spittoons and amalgam collectors.</p> 
Scope of application	Usage sur les flexibles d'aspiration chirurgicale et de salive.
Characteristics	<ul style="list-style-type: none">✓ Used daily, Dento-Viractis 99 cleans, deodorizes and avoids the formation of biofilm in just two minutes.✓ Dento-Viractis 99 does not foam and is bacteriostatic.
Commercial presentation	<p>The 1 L bottle with dosing device The 5 L bottle</p> <p>Accessories: The pump dispenser of 20 ml</p>



DENTO-VIRACTIS ASPIRATION 99

DENTO-VIRACTIS

Composition	Potassium hydroxide, non-ionic surfactant, biguanide, fragrance (apple).
Precautions	Contains D-LIMONENE. May produce an allergic reaction.
Instructions for use	Preparation of a 2% strength solution: 1. Dilute 20mL of solution for every 1L of warm tap water (40°C). Usage: 1. Circulate the solution through each aspiration tube. 2. Leave the solution to work for 2 minutes. 3. Rinse with cold tap water.

Risk phrases:

- R35: Causes severe burns.

Security phrases:

- S2: Keep out of the reach of children.
- S26: In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- S36/37/39: Wear suitable protective clothing, gloves and eye/face protection.
- S45: In case of accident or if you feel unwell seek medical advice immediately (show the label where possible).
- S60: This material and its container must be disposed of as hazardous waste.

Labelling:



Version of the sheet: 1.0 24/04/2018

CE HYGITECH
www.hygitech.com
109 bis avenue Charles de Gaulle
92200 Neuilly sur Seine - FRANCE
Tel. +33 (0)1 48 01 32 89